

Joyce Masks Sizing Gauge

Nasal Mask
JOYCE / JOYCE GEL

1 inch

INSTRUCTIONS:

- 1: Print out this gauge, ensuring that there is no scaling. Please use the square above to ensure it has printed at 100% scale.
- 2: Cut out the egg shape in the centre.
- 3: Align the top of the sizing gauge with the indent between the top of the nose and the forehead.
- 4: Use the markers to determine the correct size (blue for nasal sizes, green for full face) as shown by the diagrams.

Please note:

This gauge is a reference to aid in sizing and not an exact measurement as other factors may need to be taken into account.

Nasal Mask Measurement Area

Full Face Mask Measurement Area

XS
S
M
L
XL

S
M
L
XL

Full Face Mask
JOYCE Full Face / JOYCE Full Face GEL / JOYCE Full Face^{plus}

WEINMANN
medical technology

