


Full Face Mask Sizing Guide

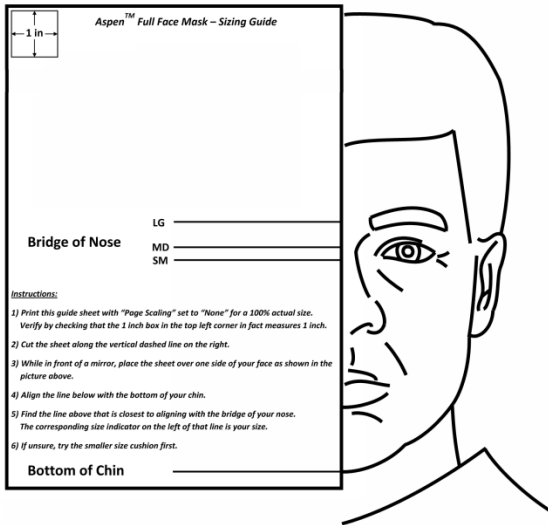

(Bridge of Nose)

LG _____

MD _____

SM _____

How to Use the Sizing Guide:

- 1) Print this sizing guide with "Page Scaling" set to "None" for a 100% actual size.
Verify by checking that the 1 inch box in the top left corner in fact measures 1 inch.
- 2) Cut the sheet along the vertical dashed line on the right.
- 3) While in front of a mirror, place the sheet over one side of your face as shown in the picture above.
- 4) Align the line below with the bottom of your chin.
- 5) Find the line above that is closest to aligning with the bridge of your nose.
The corresponding size indicator on the left of that line is your cushion size.



800-200-9842
www.innomedinc.com

(Bottom of Chin) _____