

Amara View **fitting guide**


Achieving the right fit with Amara View is easy. Simply follow the instructions below to get started.

Please read Instructions for Use for further fitting information.



1

Loosen headgear to a large setting, then disconnect the magnetic headgear clips by gripping and twisting them outward to disconnect from the magnetic headgear clip receiver.

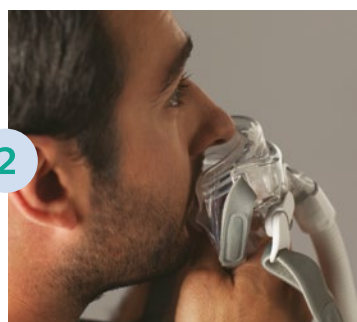

2


The mask will fit against the face straight on, under the nose and over the mouth. Place the nose over, not into, the nostril opening on top of the cushion.

Note: Do not place the mask over the nose or place the nose in the cushion nostril opening.



3

Holding the cushion in place against the face, pull the headgear over the head. Push the mask cushion up into position so that the large opening sits over the mouth and the top of the cushion fits under the nose.

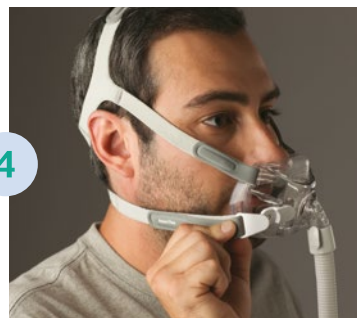

4

Connect the magnetic headgear clip to the magnetic headgear clip receiver.



5

The crown strap is very important in achieving a proper seal. Pull the crown strap tab over the top of the head to reduce leaks around the nose.



Pull back on the tabs to adjust the top and bottom side straps. The headgear should fit loosely and comfortably. Be careful not to over tighten, which is a common mistake.



Connect the flexible tubing (included with the CPAP or bi-level device) to the Amara View quick release tube.

Learn more

Find out more about Amara View full face mask. Visit www.philips.com/amaraview