

# Philips Therapy Mask System 3100 NC and 3100 SP patient-fitting guide

Follow the instructions to get started fitting Philips Therapy Mask System 3100 NC and 3100 SP

## Nasal Cradle Under-the-Nose

## Silicone Pillows



For the best performance, use the smallest cushion that fits the nose. Hold the cushion in front of the face. Check that the L and R symbols are upright and on the correct sides.



Holding the cushion against the nose with one hand, pull the headgear straps over the head.



The cushion opening will be directly under the nostrils. At no time should the nose be inside the cushion opening.



Use the sliders to adjust the top headgear straps. Pull the slider to either the left or right to tighten or loosen.



Adjust the bottom headgear straps.



The mask should fit loosely and comfortably against the face.

